

Congratulations on your new hearing instruments with the NoWax wax protection system.

NoWax helps to maintain the peak performance of your hearing instruments.

Carefully follow these instructions when changing the wax filter.

CAUTION!

- Before inserting the hearing instruments into your ear, make sure that the NoWax filter is placed correctly according to the instructions, before inserting the hearing instrument in your ear.
- The NoWax filter is not a toy and should be kept out of reach of children and anyone who might swallow or otherwise cause themselves injury.
- If the NoWax filter should accidentally work itself loose from the hearing instrument during use, please contact your hearing care provider - do not try to remove the filter yourself.
- Always visually inspect your NoWax filter before using the hearing instrument to ensure that it has not become detached from the instrument.

The filter should be changed

- when you see that the filter placed in your hearing instrument is clogged.
or
- when the hearing instrument seems weak or does not sound normal. (If this is the case, please check the filter in your hearing instrument to see if it is clogged.)

How to use NoWax

1. Remove the tool from the shell. The tool has two arms, one with the new filter and one with the removal tool.
2. Push the removal tool into the existing NoWax filter in the hearing aid and remove the used filter from the hearing aid.
3. Push the new filter on the tool into the hearing aid.
4. Throw out the used removal tool.

Be aware that you need to check and clean your hearing instrument daily, in addition to using NoWax. Cleaning instructions can be found in the Instructions for Use booklet that accompanied your hearing instrument.

